

PUTTING ON Dignity Bag™ TUBE COVER:

The best time to put on a new Dignity Bag™ Tube Cover is at the time of changing the urine drainage bag. This eliminates extra and unnecessary disconnection of the urine drainage bag tubing from the catheter, thereby minimizing exposure and contamination of the catheter.



Start with a new urine drainage bag, leave the end cap on the tubing to prevent contamination of the urine drainage bag / tubing. Push the tubing with the end cap on completely through the Dignity Bag™ Tube Cover.

Disconnect the “old” urine drain bag tubing from the catheter, remove the end cap of the new urine drainage bag tubing with the Dignity Bag™ Tube Cover previously put on,



and attach to the catheter. Spread out the Dignity Bag™ Tube Cover to hide the urine drainage bag tubing completely. Remove and launder the used Dignity Bag™ Tube Cover from the old urine drainage bag tubing so that it's ready for the next change of the urine drainage bag.



Dignity Bag™ Tube Cover can be left on until the next time you change the urine drainage bag. It can be worn in the shower—just dry by squeezing the tubing and cover in a towel. It will be damp, but will quickly air dry.



USING Dignity Bag™

Open the Dignity Bag™ Bag velcro closure at the top of the bag. Insert the urine drainage bag and spread and flatten it inside the Dignity Bag™ as much as possible.

Make sure all parts of the urine drainage bag are hidden inside the Dignity Bag™. Make sure part of the Dignity Bag™ Tube Cover is also down inside the Dignity Bag™.



With the urine drainage bag tubing placed approximately in the middle of the Dignity Bag™ opening, seal the top of the Dignity Bag™ closed around the tubing tightly with the velcro closure.



Now decide where you want to hang the Dignity Bag™ on your wheel chair, walker, etc. Use the velcro straps and loop thru the D-rings at the top of the bag. The length of the straps can be adjusted for proper length and hanging.

Remember when hanging the Dignity Bag™ that it should always be hung below the level of your bladder and minimize bending and looping of the urine drainage bag tubing to ensure proper drainage. Also, make sure the placement of the Dignity Bag™ and tubing is such that there is no danger of getting



caught or tangled in any protruding or moving parts. The only time the Dignity Bag™ needs to be removed is for showering and laundering. Leave it on for privacy and protection of the urine drainage bag from tears and dirt.



REMEMBER

With the urine drainage bag out-of-sight, remember to check and empty periodically to avoid the danger of an overly full drainage bag.

LAUNDERING

Dignity Bag™ and Tube Cover can be washed with like colors in regular detergent in any water temperature. Fasten velcro parts together prior to washing and drying to minimize velcro from sticking to other garments. **DO NOT BLEACH.** Line dry or tumble dry low to medium heat, squeezing the tubing and cover in a towel. It will be damp, but will quickly air dry.